



Introducing the Symmetry Access™ Low Profile Retractor

- Ultra-low profile
- Contours to patient anatomy
- Patented single-use technology
- Optimizes exposure in small incision surgery

A close-up photograph of the Symmetry Access Low Profile Retractor. The device is made of polished metal and is shown in a curved, open position. It is set against a dark background with a blurred surgical instrument in the background. The lighting highlights the metallic texture and the sharp edges of the retractor.

Symmetry Access™
Low Profile Retractor

Symmetry Access™ Low Profile Retractor

An ultra-low profile approach to small incision soft tissue retraction

Designed to Optimize Exposure for Better Surgical Visualization


- Low profile aspect reduces instrument interference
- Anatomy-contouring application minimizes shadows

Designed to Allow Freedom in Retraction

- No frame needed – Stable around small incisions
- Can be applied to skin or drapes
- Independently adjustable for optimal retraction tension
- Polycarbonate tip retractors do not need to be removed prior to imaging

Designed for Ease-of-Use

- Simple application
- Easy repositioning

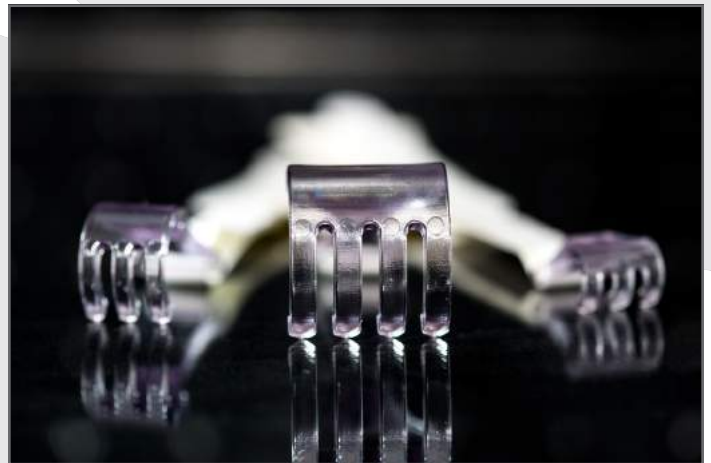


Range of Specialized Tips




Symmetry Access™ Low Profile Retractor

Every aspect of the Symmetry Access™ Low Profile Retractor has been optimized for lower profile, soft-tissue retraction.

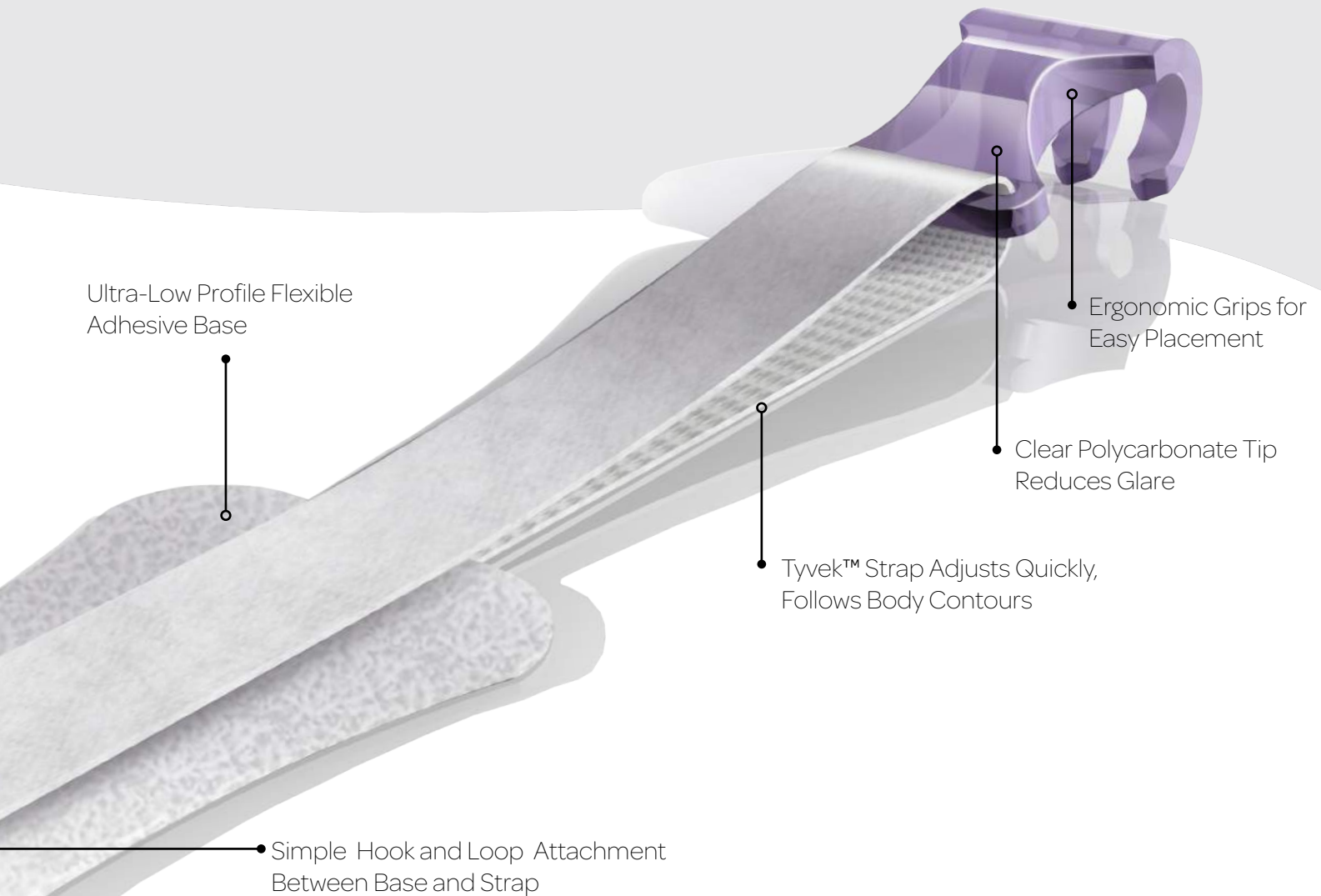


Symmetry Access™ Low Profile Retractor

1

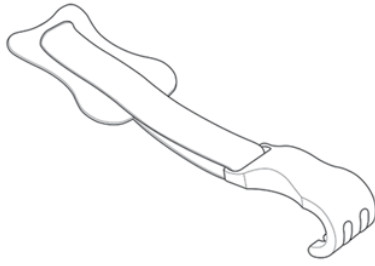


Open the pouch, peel the backing from the adhesive base, and place on a clean, dry part of the patient or on surrounding drapes. The base will adhere securely.

2

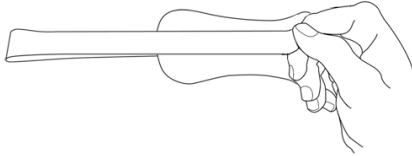

Use the simple hook and loop strap attachment to establish exactly the retraction tension required. Use as many retractors as you need.

3

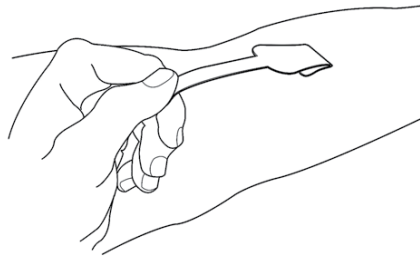

Detach the retraction strap, remove the base from the patient's skin by peeling low and slow, and then dispose of the retractor.

3PAW SMALL	3PAW MEDIUM	4PAW LARGE	5mm HOOK SHARP	5mm HOOK BLUNT

A range of sizes and tips allows use of the Symmetry Access™ Low Profile Retractor for a variety of small incision, soft-tissue procedures.

Visit symmetrysurgical.com or contact your local Sales Representative for more information.

Symmetry Access™ Low Profile Retractor

CODE	DESCRIPTION	QTY
RTK-3PAWS	Low Profile Retractor, Single Use 3 Prong Small	20 sterile retractors/box
RTK-3PAWM	Low Profile Retractor, Single Use 3 Prong Medium	20 sterile retractors/box
RTK-4PAWL	Low Profile Retractor, Single Use 4 Prong Large	10 sterile retractors/box
RTK-HTIPS5	Low Profile Retractor, Single Use Sharp Hook	20 sterile retractors/box
RTK-HTIPB5	Low Profile Retractor, Single Use Blunt Hook	20 sterile retractors/box

